## Ezox®

Medical device for the treatment of gastro-oesophageal reflux

**INDICATIONS:** EZOX® is a mechanical medical device that allows rapid relief from the symptoms of gastro-oesophageal reflux: pyrosis, epigastric pain, acid reflux, irritative cough and dysphonia.

Gastro-oesophageal Reflux Disease (GERD), is characterised by symptoms (pyrosis, acid reflux, irritative cough and dysphonia) with or without concomitant erosion of the oesophageal mucosa.

GERD is usually exacerbated when lying down and symptoms can therefore be worse at night time.

EZOX® is an original and innovative hyaluronic and chondroitin sulphate complex, that protects the gastro-oesophageal mucous membrane.

The inclusion in the formulation of EZOX® of a highly bioadhesive component (poloxamer 407), allows the hyaluronic acid and chondroitin sulphate to adhere to the wall of the oesophagus for longer, to provide better protection against the hydrochloric acid produced by the stomach.

Chondroitin sulphate also favours gastro-oesophageal mucosal repair processes.

**COMPOSITION:** Hyaluronic acid, chondroitin sulphate, polyvinylpyrrolidone, xylitol C, sodium benzoate, potassium sorbate, flavourings, purified water, poloxamer 407.

**PACK:** 20 stick packs of 10 ml.

**DOSE AND DIRECTION FOR USE:** 10 ml of solution after main meals and at bedtime, or as instructed by your doctor.

**CONTRAINDICATIONS:** There are no known side effects for the product. It is recommended to avoid use in the case of known or suspected hypersensitivity to any of its components. Furthermore, since no specific studies have been performed on pregnant or breastfeeding women, the use of the product is not advisable in such cases. Consult a doctor before use.

**WARNINGS:** not to exceed the recommended dose. To contact a doctor if symptoms persist or side effects occur. Not to use the product after the expiry date. To store the product in a cool, dry place, away from sources of heat. Not to freeze the product. To keep the product out of the reach of children.

**Manufacturer:** Apharm s.r.l – Via Roma, 26 – 28041 Arona - Italy

**Registration holder:**

Megapharm Ltd. 8 Hapnina St. Raanana Zip code: 4321545, Telephone: 09-7604596

Ministry of Health medical device license no. 27570006

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